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HSV Typ RQ Kit

Manual

For Real-Time PCR Detection and
Differentiation of Herpes Simplex
Type 1 and 2

For use with Rotor-Gene and StepOne
Research use only

NG-WI-ASL-07-200
Version 2.0
Summer 2021

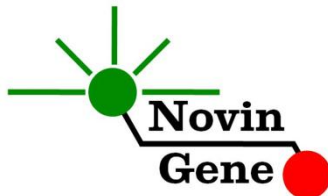
Table of Contents:

1. Introduction	2
2. Kit Contents	3
3. Storage and Stability	3
4. General Precautions.....	3
5. Additionally Required Materials.....	4
6. Specimen, storage and transport.....	4
7. Interfering Substances	5
8. DNA isolation.....	5
9. Internal control (IC).....	5
10. PCR Protocol.....	6
11. Programming of the Rotor-Gene	6
12. Programming of StepOne(/Plus)	7
13. Programming other machines	7
14. Data Analysis: Rotor-Gene.....	8
15. Data Analysis: StepOne	10
16. Sensitivity.....	13

HSV Typ RQ kit is intended for the detection and typing of HSV-1 and HSV-2 DNA with Rotor-Gene or StepOne machines. This kit is for research use only.

1. Introduction

Human Herpes Virus type 1 and 2 (HHV-1, HHV-2) known as Herpes simplex virus 1 and 2 (HSV-1, HSV-2) are large enveloped DNA viruses with a genome of about 150 kbp double stranded DNA.

HSV is a ubiquitous agent and can cause a variety of infections. Although most HSV infections in adults are usually benign, yet HSV is the most common cause of viral encephalitis and genital herpes is one of the most prevalent sexually transmitted diseases (STDs). Despite overlapping clinical manifestations, HSV-1 and HSV-2 infections are different regarding the severity and outcome, mortality, frequency of reactivation, response to medication, and drug resistance. Therefore, defining the type is clinically important and routinely recommended.

Classical HSV-1 and HSV-2 infections are associated with oral or genital diseases, respectively. However, the differentiation of HSV-1 from HSV-2 based on anatomical site of infection is far from absolute and both HSV-1 and -2 can be isolated from skin, mucosa, and visceral organs. Also HSV-1 has become the predominant cause of genital ulcers in some developed countries. Therefore, HSV1 and HSV-2 detection and typing based on clinical symptoms is not reliable and requires laboratory tests.

Also in case of viral encephalitis, prompt laboratory diagnosis of HSV infection is essential for patient management and possible initiation of antiviral therapy and reducing the mortality rate.

HSV Typ RQ kit serves the both of above, fast and highly sensitive detection and typing of HSV-1 and HSV-2 infections based on Real-Time PCR. In this method application of fluorescent probes

allows detection and discrimination of amplified product without requiring post-amplification analysis, therefore, reducing the possibility of contamination with the PCR product. This kit is designed for Rotor-Gene or StepOne machines. This kit also incorporates an Internal Control (IC) to evaluate DNA extraction quality or possible PCR inhibition.

2. Kit Contents

The kit contains a manual, a CD with Rotor-Gene and StepOne templates and following reagents:

Label	Content	Quantity
HSV Typ Mix	PCR Master mix*	360 µl
HSV-1 Ctrl	HSV-1 Positive Control	150 µl
HSV-2 Ctrl	HSV-2 Positive Control	150 µl
HSV Type IC	Internal Control*	250 µl
Water	PCR Grade Water	200 µl

* 1, 2 and 4 tubes for 24, 48 and 96 reaction kits.

3. Storage and Stability

The kit components should be shipped and stored at -20°C and are stable until the expiry date mentioned. Avoid repeated freeze-thaws more than few times to prevent reduced sensitivity.

4. General Precautions

In order to prevent false results, always pay attention to the following points:

- Treat all samples as potentially infectious.
- Within the pre-PCR work area assign three separate spaces for: a) Sample storage and extraction, b) Reagent preparation where the master-mix is aliquoted into tubes

and c) Reaction preparation area for addition of extracted DNA to the tubes.

- Always wipe the working surfaces with 70% Ethanol before and after work.
- Thaw kit components on **crushed ice** completely, mix by flickering followed by a quick spin and **store on crushed ice while working**.
- Do not place PCR tubes on crushed ice. Use cold blocks instead.

5. **Additionally Required Materials**

To use this kit, you need the following items:

- Real-Time PCR machine and accessory computer
- Table top microtube centrifuge
- Vortex Mixer
- Dry Block Heater
- Adjustable pipettors and nuclease free filtered tips
- DNA extraction kit
- Nuclease free 1.7ml microtubes and PCR microtubes
- Disposable powder-free gloves
- Cold block

6. **Specimen, storage and transport**

CSF or swab of the genital lesions should be collected in sterile condition in proper and sterile tubes. Samples should be shipped at +4°C. Upon receipt sample can be stored at +4°C for few days or stored at -20°C for up to few weeks.

7. Interfering Substances

Heparin (more than 10 IU/ml) affects the PCR. Blood collected in heparin containing tubes should not be used. Samples of heparinized patients must not be used as well.

Elevated levels of bilirubin (≤ 4.5 mg/dl) and lipids (≤ 1000 mg/dl) and hemolytic samples do not influence the extraction and PCR.

8. DNA isolation

DNA isolation can be performed using different kits from various manufacturers. We recommend the following:

- High Pure Viral Nucleic Acid Kit (Cat. no. 11858874001, Roche Applied Science, Mannheim, Germany)
- QIAamp DNA Blood Mini Kit (Cat. no. 51104, Qiagen GmbH, Hilden, Germany)
- QIAampUltraSens ® Virus Kit (Cat. no. 53704, Qiagen GmbH, Hilden, Germany)
- QIAampMiniElute Virus Spin Kit (Cat. no. 57704, Qiagen GmbH, Hilden, Germany).

To monitor DNA extraction process, internal control should be applied to the extraction process. For more details, please, refer to section 9 of this handbook.

9. Internal control

In order to evaluate the possibility of DNA extraction failure and PCR inhibition and prevent false negative results, HSV Type RQ kit contains internal control (IC). IC can be used during extraction process or simply added to HSV Typ Mix. To monitor both DNA extraction and PCR reaction, internal control should be added to the mixture of lysis buffer and patient sample. Required volume of internal control is 10% of elution buffer. For instance if extracted DNA is eluted with 100ul, then 10ul of internal control should be

added to the mixture of lysis buffer and patient sample. **Please note that internal control should not be added directly to the patient sample (i.e. before addition of lysis buffer) as it loses its efficiency.** If IC is added to HSV Typ Mix, only PCR inhibition can be monitored. For this purpose, 1ul of IC reagent should be added to each reaction. For example, for 10 PCR reactions, 15ul of internal control should be added to 150ul of HSV Typ Mix before it is added to the tubes.

In a successful DNA extraction and PCR test, Internal control should generate a CT of 26-34 in Rox/Orange Channel.

10. PCR Protocol

Thaw the reagents on ice completely followed by a brief mixing and a quick spin. Place required number of tubes on cold block. Consider one tube for each sample plus two for Positive controls and one for the Negative control.

If IC is introduced during extraction process, pipette 15µl of HSV Typ Mix to each PCR tube.

If IC is added to the HSV Type Mix, add 15ul of the prepared mix (as described in section 9) to each PCR tube.

Then add 10ul of extracted DNA, Controls or water to each tube.

Cap the tubes and visually inspect to make sure all are capped securely. Place tubes in the machine.

Note: Working with StepOne instrument, spin tubes briefly before loading on the block.

Note: If using Rotor-Gene attach the locking ring.

11. Programming Rotor-Gene

Before you start the machine make sure you have attached the locking ring on the rotor!

Open the CD provided in the kit and double click on HSV Typ 0.2 or HSV Typ 0.1 according to the used tubes. Program starts. Click on Start button (Green button on the top menu). On the pop up window click start again and save the run file.

12. Programming StepOne

Open the StepOne software (V 2.*). On the Set Up menu click on Template and select the file on CD provided with the kit. Click on Plate Setup. Positive and Negative controls and few samples are defined. You may change plate set up using right click options (copy, paste, clear). You may also add/remove samples or change sample names on “Define Targets and Samples” menu. When finished, click on “Start Run” and save the experiment. Instrument will start shortly.

Note! ROX should not be selected as reference dye in this test.

13. Programming Other Machines

If you apply this kit to other Real-Time PCR machines, program it according to the following table:

Step	Temperature and time	Cycles
1	95°C x 10 min	1 cycle
2	95°C x 15 sec	45 cycles
	60°C x 40 sec	

Fluorescence should be collected at 60°C for FAM, VIC and ROX dyes.

Note! ROX should not be selected as reference dye in this test.

14. Data Analysis: Rotor-Gene

Before analyzing results, make sure in the sample menu the Positive controls have been defined as “Positive control”. Patient samples should be defined as "unknown" and Negative control or no template control as "Negative Control" or "NTC" respectively. Analyze data according to manufacturer recommendations. Perform qualitative analysis for **HSV-1 on Green channel**, **HSV-2 on Yellow channel** and for **Internal Control on Orange channel**. Briefly, click on analysis menu and then under Quantitation tab double click on cycling A. Green. Set the threshold at 0.1. Repeat the above for Cycling A. Yellow and Orange. Figures 1, 2 and 3 represent typical graphs for Rotor-Gene machine.

Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used.

In the absence of sigmoid graph and log phase, sample is considered Negative and CT if present is not reliable.

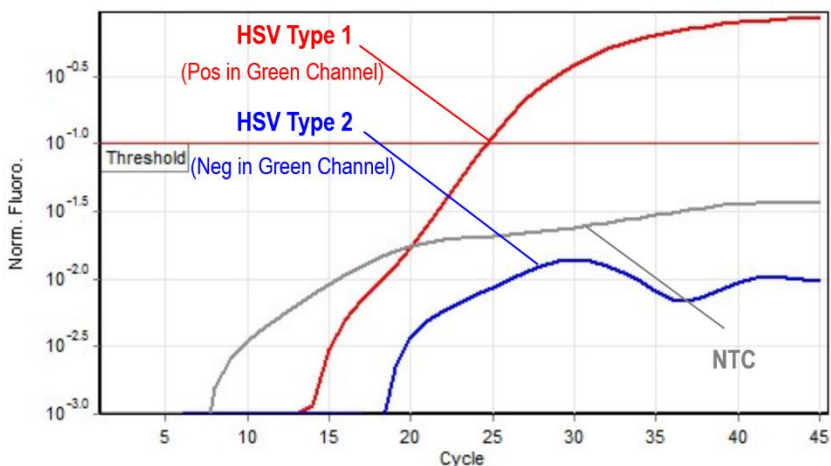
Fig 1. Typical HSV controls graph in Green channel for Rotor-Gene

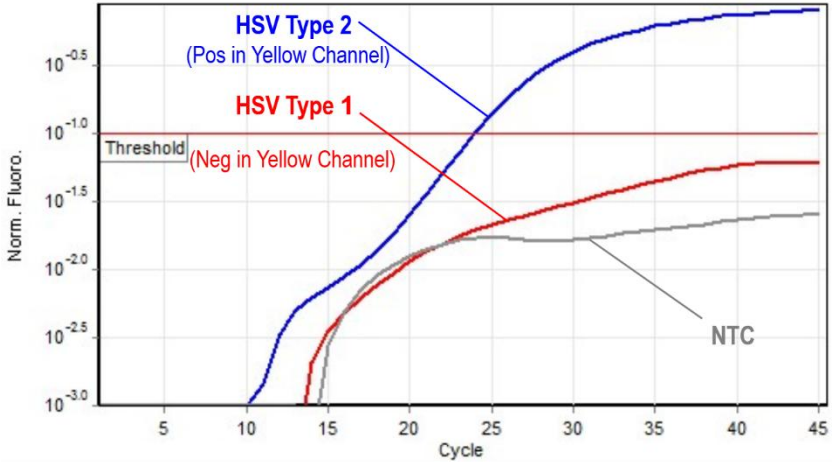
Fig 2. Typical HSV Controls graph in Yellow channel for Rotor-Gene

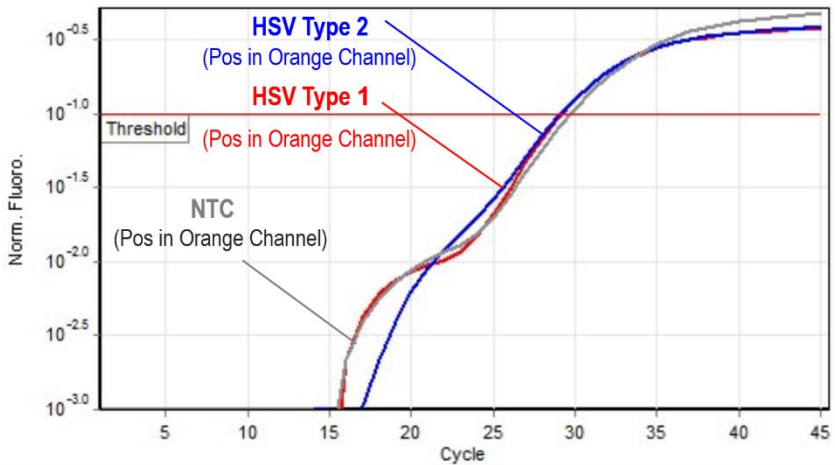
Fig 3. Typical HSV Controls graph in Orange channel for Rotor-Gene

Consider following points when analyzing:

- A sample is **Positive** for **HSV-1** if it is positive in Green channel with a CT of less than 40.
- A sample is **Positive** for **HSV-2** if it is positive in Yellow channel with a CT of less than 40.
- A sample is **Negative** if it is negative in both of Green and Yellow channels while it is positive for IC in Orange channel with CT of 26-34.
- Results are **Inconclusive** and the test should be repeated if sample is negative in all three channels of Green, Yellow and Orange.

15. Data Analysis: StepOne

Analyze data according to manufacturer recommendations. Briefly, click on Analyze and set the threshold for **HSV-1/FAM** at 2,000; for **HSV-2/MIC** at 1,000; and for **IC/ROX** at 200.

Figures 4, 5 and 6 represent typical graphs for StepOne machine.

Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used.

In the absence of sigmoid graph and log phase, sample is considered Negative and CT if present is not reliable.

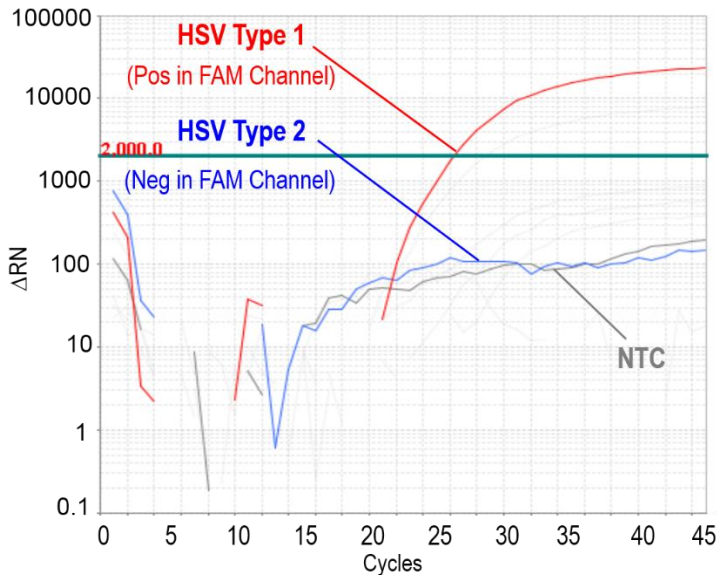
Fig 4. Typical HSV Controls graph in FAM channel for StepOne

Consider following points when analyzing:

- A sample is **Positive** for **HSV-1** if it is positive in FAM channel with a CT of less than 40.
- A sample is **Positive** for **HSV-2** if it is positive in VIC channel with a CT of less than 40.
- A sample is **Negative** if it is negative in both of FAM and VIC channels while it is positive for IC in ROX channel with CT of 26-34.
- Results are **Inconclusive** and the test should be repeated if sample is negative in all three channels of FAM (HSV-1), VIC (HSV-2) and ROX channel (IC).

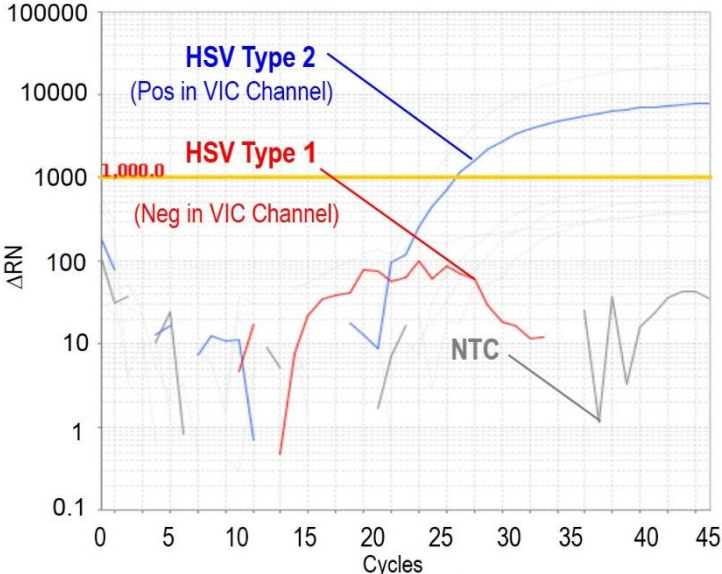
Fig 5. Typical HSV Controls graph in VIC channel for StepOne

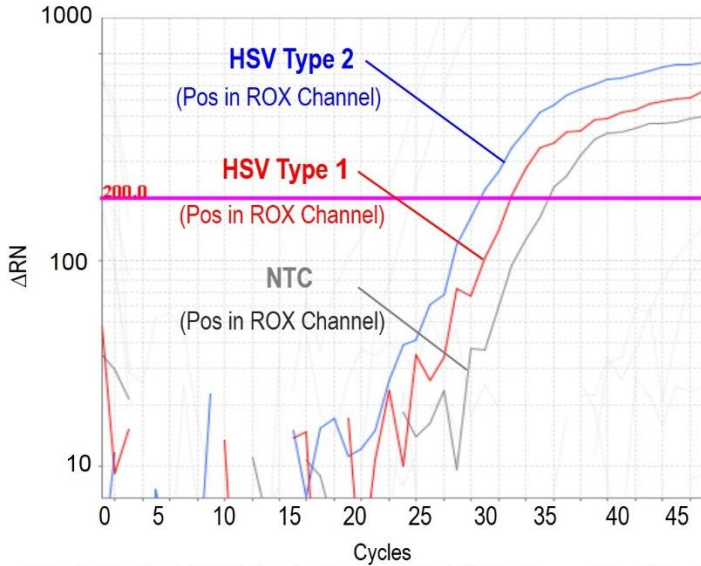
Fig 6. Typical HSV Controls graph in ROX channel for StepOne

16. Sensitivity

The analytical detection limit of the kit was assessed with dilution series of the target sequence and showed a limit of detection equal to 0.5 copy/ul.